

RETYPEEEL®




The new-generation
biorevitalizing and lifting peel

Trichloroacetic Acid | Mandelic Acid | Kojic Acid

The heart of **REVYPEEL®** technology

REVYPEEL® is a Class IIa medical device designed to achieve dermo-epidermal stimulation of fibroblasts without irritating the superficial layer.

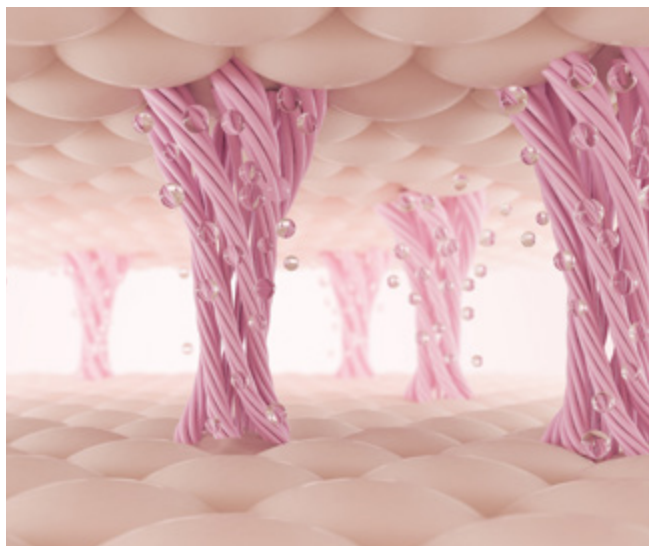

Effectiveness

Thanks to the synergistic combination of **trichloroacetic acid (TCA)**, **mandelic acid** and **kojic acid**, **REVYPEEL®** effectively treats:

- signs of **photo-aging** and **chrono-aging**
- **acne lesions** and **scars**
- **skin dyschromia** and **pigment spots**
- **stretch marks** and **irregularities** in **tone** and **texture**

Suitable for all ages, **REVYPEEL®** does not require needles, does not cause frost, and produces no downtime – a clinical advantage that patients immediately understand.

MADE IN ITALY



A single formula, two specific concentrations: **REVYPEEL LOW and REVYPEEL HIGH**

REVYPEEL HIGH

Deep stimulation for visible and long-lasting results

A superficial chemical peel with a **higher concentration of TCA**, capable of acting across the entire thickness of the epidermis.

Specifically formulated to treat:

- hypertrophic keloids and scars
- acne scar outcomes
- significant dyschromia and melasma
- stretch marks and deep scars

By removing the most damaged skin layers, **REVYPEEL HIGH stimulates deep cellular regeneration**, reducing wrinkles, scars and pigment spots, restoring a **more compact, even and radiant skin**.

REVYPEEL LOW

Superficial action, deep stimulation

A superficial chemical peel acting on the **stratum corneum and granular layer**, of the epidermis, indicated for:

- mild acne lesions
(papules, pustules, nodules, cysts)
- light dyschromia
- fine wrinkles and irregular texture

The synergy of its acids **stimulates cellular turnover, reduces superficial scars and evens out the complexion**, without inducing a frost effect.

REVYPEEL LOW acts as a **non-injectable biorevitalization**, activating a **natural regeneration process**. It can be used alone or combined with other medical-aesthetic treatments.

COMPOSITION

TRICHLOROACETIC ACID
(20%)

MANDELIC ACID

KOJIC ACID

EXCIPIENTS



COMPOSITION

TRICHLOROACETIC ACID
(12%)


MANDELIC ACID

KOJIC ACID

EXCIPIENTS

Active ingredients

An effective synergy

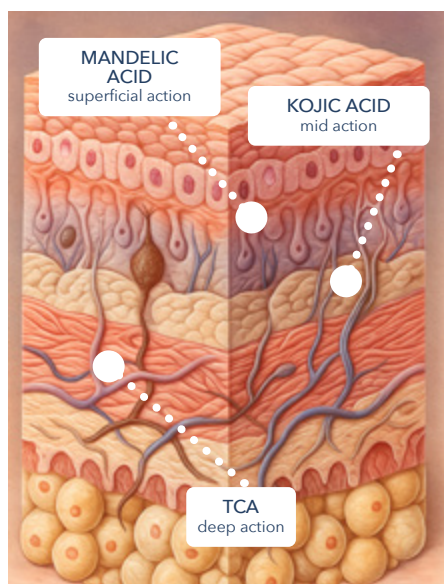

Mandelic Acid

Hydrating, regenerating, bactericidal, nourishing. Promotes the production of collagen and elastin, improving tone and texture.

Trichloroacetic Acid (TCA)

Keratolytic, brightening, stimulating, bactericidal. Promotes controlled exfoliation and the synthesis of new collagen, enhancing firmness and radiance.

Kojic Acid

Brightening, depigmenting and hydrating. Acts as a powerful tyrosinase inhibitor, reducing melanin synthesis and countering dyschromia.

Strengths

- Needle-free, **non-invasive and painless**
- **No "frost" effect**
- **Non-photosensitizing**, usable all year round (LOW)
- **Suitable for all phototypes (I-VI)** and all ages
- Stimulates **tissue regeneration and remodeling**
- Provides **luminosity, firmness and uniformity**

To be performed exclusively by medical personnel or under medical supervision

Clinical indications

CHRONO-AGING

MAKES TISSUES FIRMER,
SMOOTHER AND BRIGHTER

DYSCHROMIA

LIGHTENS SKIN SPOTS
BY INHIBITING TYROSINASE

PHOTO-AGING

REDUCES SPOTS AND DYSCHROMIA

SCARS

SMOOTHING AND RESTRUCTURING

ACNE

SOOTHING, ANTI-BACTERIAL AND
ANTI-INFLAMMATORY ACTION

STRETCH MARKS

IMPROVES ELASTICITY
AND SKIN APPEARANCE

REVYSKIN

Complementary Line



Pre-Peeling Lotion

Balances skin pH,
preparing the skin for the
peeling action



Neutralizing Gel

Alkaline formula (pH 9)
that blocks and neutralizes
REVYPEEL® 's chemical action.



Post-Peeling Cream SPF 50+

Protects, soothes and regenerates
the skin after treatment.
Prevents spot formation and
photo-aging damages.

Treatment protocol

1

PREPARATORY SOLUTION

Balance skin pH before application.

2

APPLICATION OF

REVYPEEL®

Apply evenly and massage vigorously to promote active penetration.

3

NEUTRALIZING SOLUTION

Dab with the specific gel to stop the chemical action.

4

POST-PEELING SOOTHING CREAM

Promote hydration, protection and immediate comfort.

Visible effects after the first session:

BIO-REVITALIZING

BRIGHTENING

LIGHTENING

BACTERICIDAL

LIFTING



Clinical Results

Regular use of **REVYPEEL®** allows you to:

- Counteract photo-aging and chrono-aging
- Reduce fine and deep wrinkles
- Revitalize the dermis and epidermis
- Brighten and even out the complexion
- Increase skin hydration
- Improve acne and scars
- Reduce stretch marks and melasma

Synergies

REVYPEEL® can be used as a **stand-alone treatment** or in **synergistic combination with other medical-aesthetic protocols** to enhance effectiveness.

COMBINED TREATMENT

WHEN TO USE REVYPEEL®

Injectables	Before
Botox	Before
Microneedling	Before
Topical Treatments	Before
Fractional Radiofrequency / Laser	Before
Carboxytherapy / Oxygen	Before
PDO Threads	Before
Cryotherapy	Before
PRP	Before
Nanoneedling	After
IPL	After

Average Product Consumption per Treatment

To support clinicians in proper protocol planning, below are the **average quantities** typically used for one **REVYPEEL®** session based on clinical indication.

PRODUCT	FACE/NECK/ DÉCOLLETÉ	ACNE	SCARS	KERATOSIS*	STRETCH MARKS*
REVYPEEL LOW	1.5 ml - 2 ml	2 ml	–	1.5 ml	–
REVYPEEL HIGH	1.5 ml	–	1.5 ml	1 ml	3.5 ml
Pre-Peeling Lotion	5 ml	5 ml	5 ml	5 ml	5 ml
Neutralizing Gel	5 ml	5 ml	5 ml	5 ml	10 ml
Post-Peeling Cream	2 ml	2 ml	2 ml	2 ml	4 ml

** Quantities may vary based on the size of the area treated.*

These values represent an indicative estimate, useful to accurately plan the number of treatments per vial and define patient therapeutic cycles in advance.

REVYPEEL® represents a new frontier in medical skin rejuvenation, combining chemical potency and gentle action in a single procedure.

A safe, versatile, year-round treatment capable of deeply regenerating the skin and restoring authentic, firm and radiant beauty.



Head Office: Via Castiglione 99, 40136 BOLOGNA
Warehouse: Via Bruno Tosarelli 356, 40055 Villanova di Castenaso (BO)
www.mbemedicale.it - info@gtrading.it



MBE is a G. TRADING COMPANY BRAND
REVYPEEL is a registered trademark of G. Trading Company.

Disclaimer: All contents of this material are for informational purposes only and are provided by G. TRADING COMPANY without any warranties of any kind. Healthcare professionals are solely responsible for making their own independent evaluation regarding the suitability of any product for any particular purpose and in accordance with country-specific regulations. The availability of products and the indications mentioned in this material are subject to the regulatory requirements and product registration status in each country. Please refer to your User Manual/Protocol for country-specific indications. Products and technical specifications may change without notice. For more information, please contact G. TRADING COMPANY.
© 2025 G. TRADING COMPANY. This material contains registered and unregistered trademarks, trade names, service marks, and brand names of G. TRADING COMPANY and its affiliates. All other trademarks are the property of their respective owners. All rights reserved.